## HS-200M HEALTH SCALE


The Health Scale HS-200M is designed to both weigh the body and measure the height and meets the OIML International Recommendation R76. The scale has the advantage of a large weighing range, accuracy and high sensitivity. It is ideal for school nurse's offices or for doctor's screening rooms where height and patient weight measuring is necessary. It is also can be used for other weighing needs.

### Specification:

1. Maximum capacity: 160kg/350lb, 180kg/400lb, 200kg/450lb;
2. Graduation: 100g/0.25lb
3. Height Range: 60-210cm
4. Minimum value of height per division: 5mm
5. Platform size: 375 cm x 275 cm
6. Overall dimension (L x W x H) 530mm x 275mm x 1485mm

### Installation Instructions:


Installation Video Link: <https://www.brecknellscales.com/resources/videos/>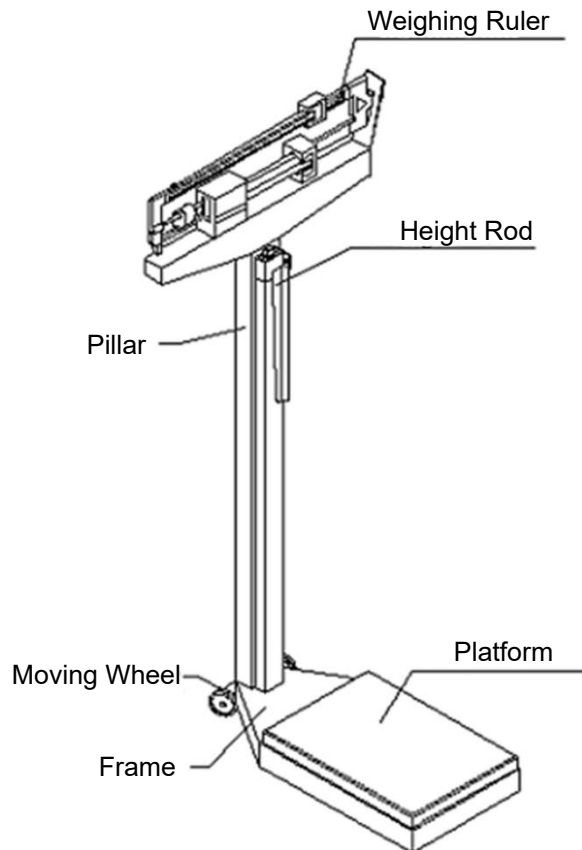

Installation Video





---

Pillar Installation (Fig.2):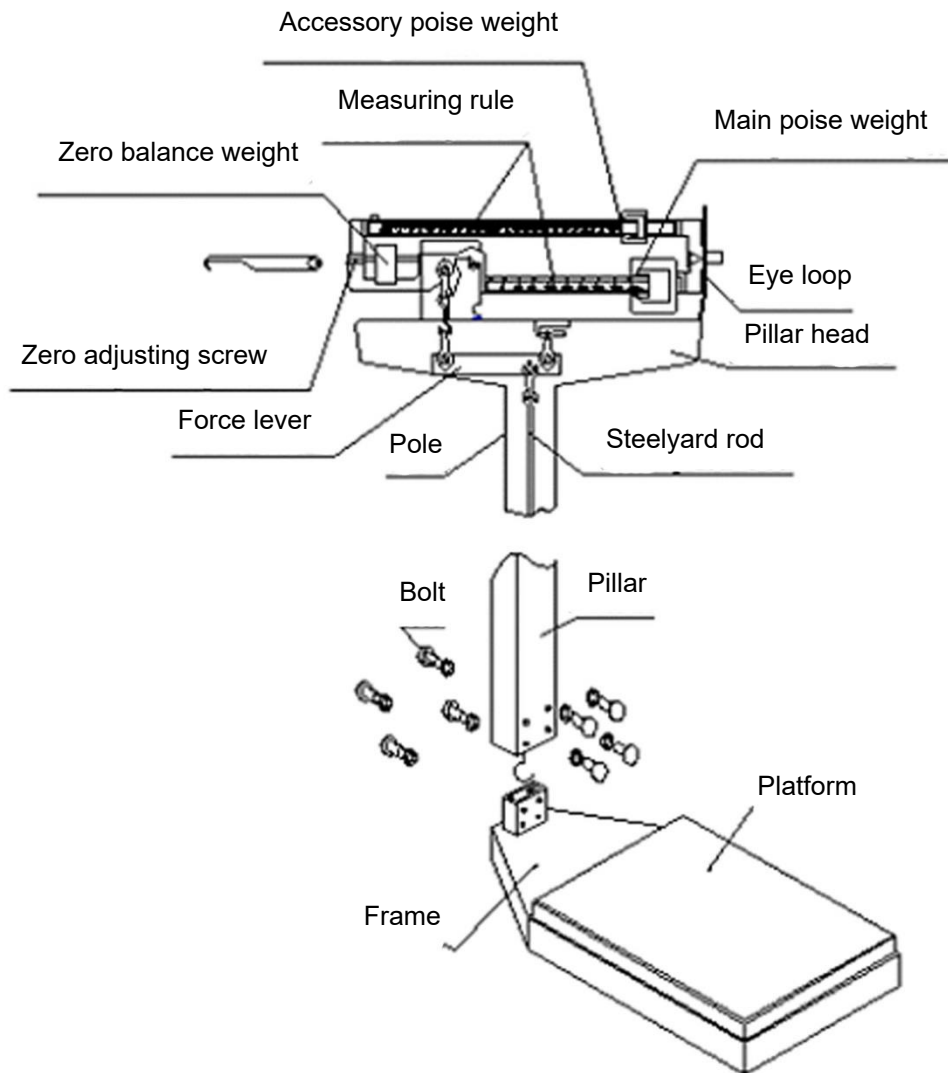

1. Insert the pillar upright to the bracket on frame
2. Use the screws and washers to fix the pillar in a vertical position

II. Connect the steelyard rod to long lever (Fig.3):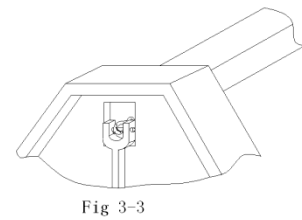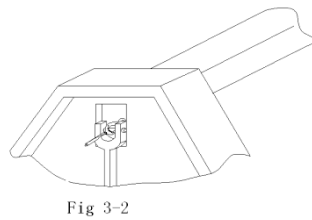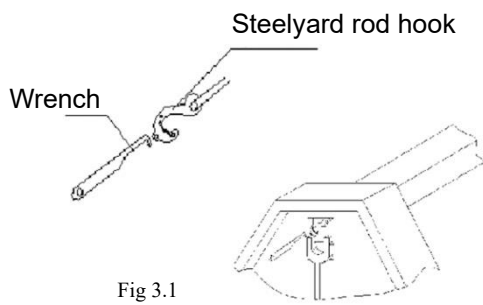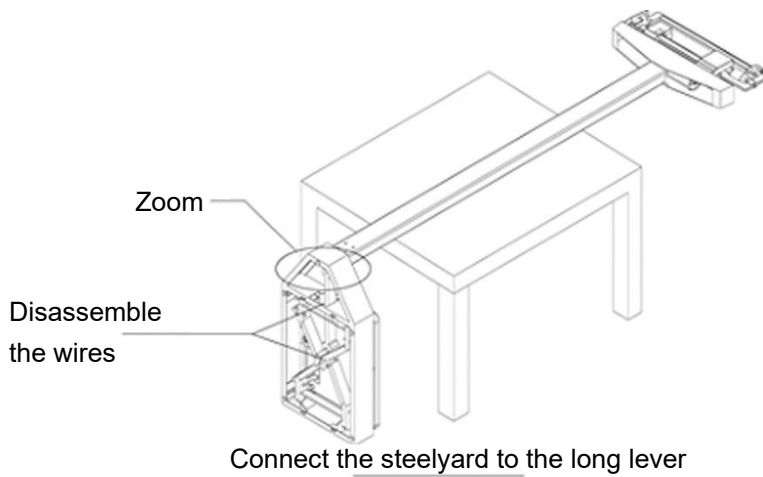

1. Lay the pillar on a table (Fig.3)
2. Disassemble the wires on the levers
3. Insert the wrench into the small hole in front of the steelyard rod, and pull the hook of the steelyard rod with the hook of wrench (Fig. 3.1)
4. Push the long lever forward, then hook the bearing of the steelyard rod on the pivot of the long lever (Fig. 3-2)
5. Take away the wrench from the draft rod (Fig.3-3)
6. Put the scale on the ground gently

### III. Height Rod Installation:

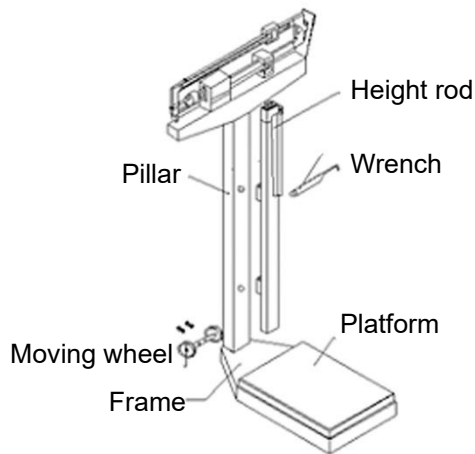
Fig 4: Height rod installation

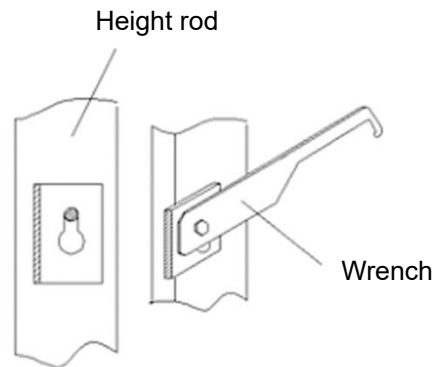
Fig 4.1: Back side hole of the height rod

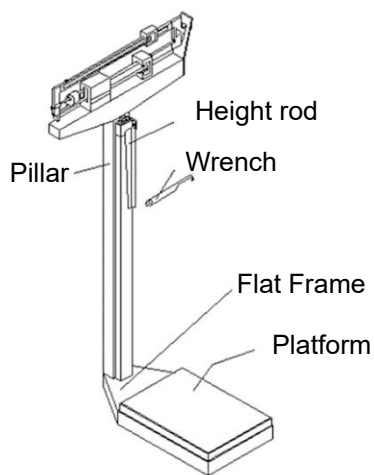
Fig 4.2

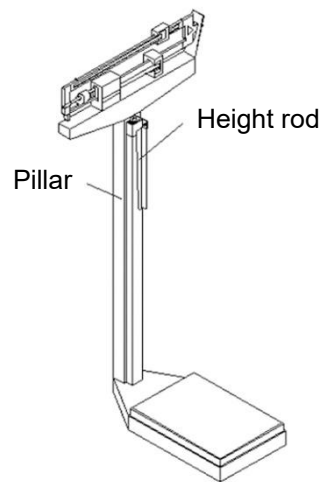
Fig 4.3

1. Insert the back side holes of the height rod into the two bolts on the front side of pillar; loosen the height rod pulling downwards until in correct position. (Fig.4-1)
2. Use the wrench to tighten the two hex-head screws. Do not over tighten the screws.
3. Remove the wrench from the two hex head screws. (Fig.4-3)

---

IV. Moving Wheel Installation (Fig.5):

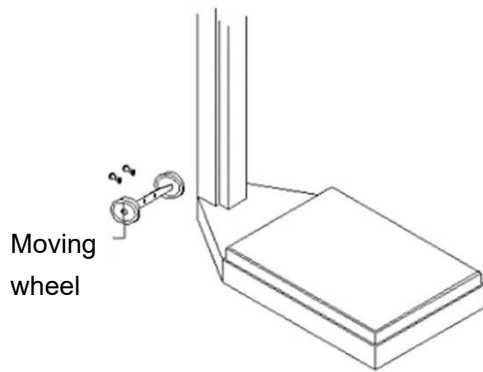
Fig. 5

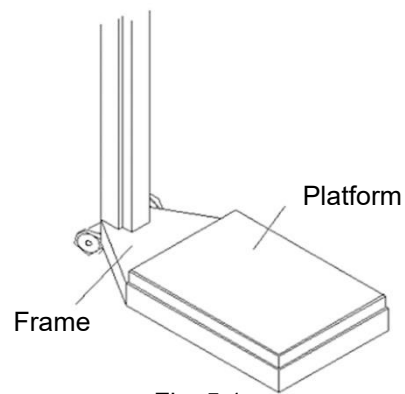
Fig. 5.1

1. Turn the wheels so that the holes are facing the rear of the scale, place the screws in the holes. (Fig.5-1)
2. Use the screws and washers to fix the wheel to the platform, adjust the angle iron to a level position and then tighten the screws.
3. When moving the scale, make sure the base of the scale is lifted from the floor, do not drag the scale.

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