Model BS-713

Digital Body Scale

Instruction Manual

P/N: AWT35-501471
Issue AA

Power Supply

- Two CR2032 lithium batteries

NOTE: Please take out the insulation tape before use.

Operation

1. Press the **Set** button to turn on the scale or step on the platform.
2. Use the **kg/st/lb** button on the bottom side of the scale to select a unit of measure.
3. Press the **Set** button to choose your personal code when the display shows 0.0kg. Use the arrow buttons to select a number from P0 to P9. Press the **Set** button again.
4. The display will flash. Use the arrow buttons to enter your height in cm or feet/inches. Press the **Set** button to confirm.
5. Place the scale on a flat, level surface. Do not use on carpet as it may affect accuracy.
6. Step onto the platform. Once the weight has stabilized, body weight will be locked on the display.
7. The scale will turn off automatically after 10 seconds of non use.

Error Codes

L  - This is a lower battery indication. Open the battery cover and replace the batteries observing the correct polarity.

E r r - Maximum capacity has been exceeded (over 180kg / 396lbs).

Care Instructions

- Clean the scale with a damp cloth. **Do not allow water to seep into the unit and scale body. Never use abrasive cleansers or chemical solvents.**
- Remove the battery from the scale if it has not been used for a long period of time.
- Do not stack anything on the scale during non use.
- Do not put the scale in an upright position.
- Do not drop the scale as this may damage the sensor and glass.
- There are no user replaceable parts. Do not open the scale case.

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